



# Scour Test Kit

Identify pathogens in calf manure



A simple and accurate cassette style test kit, to identify pathogens in calf manure.



5 x tests in each kit

- Kit tests for **Rotavirus, Coronavirus, E.coli (K99) and Cryptosporidium.**
- Quick and easy to use, with results available in less than 10 minutes.
- Each kit contains 5 tests.

Product Code: **BIOK 394**

## How to proceed

- 

Take the faeces directly from the rectum of the calf. If the samples are liquid, take a spoonful.
- 

If faeces are solid, remove the excess amount using a spatula.
- 

Dilute the spoon's contents in the liquid of the small tube called the sample tube.
- 

Shake the sample tube to homogenize.
- 

Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.
- 

Remove the tube's protective cap.



## Precautions for use

- Store kits in a dry place at room temperature.
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.



7

Break off the blue tip.



8


Place the plate on a completely level surface. Deposit three drops in each small hole. Leave the device on a flat surface and wait for 10 minutes. The liquid contained in the sample tube moves to the strip and slowly migrates along the strips.



9

After 10 minutes, read the results using picture 10 as a reference.

10

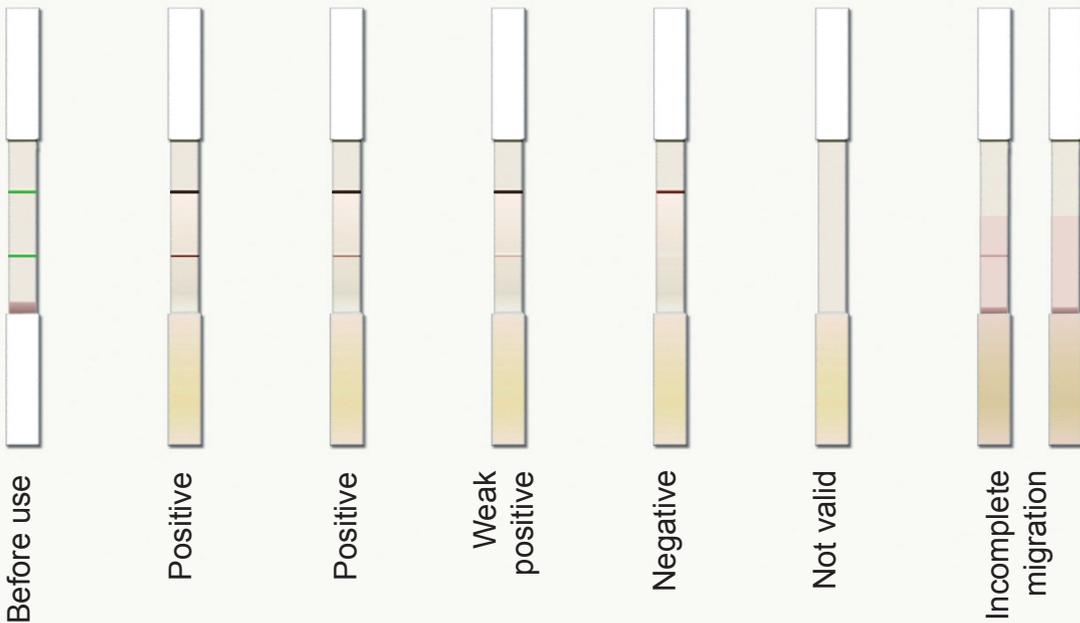

1 line = negative    2 lines = positive

